

3M™ Tegaderm™ Contact Non-Adherent Contact Layer

Protects the wound bed while reducing pain of dressing changes



Tegaderm™ Contact Layer provides a unique concept in wound management, rapidly transferring wound exudate into an absorbent secondary dressing. Clinical studies show Tegaderm™ Contact Layer can be left on the wound for up to seven days.

Key Benefits

- Minimizes disruption of granulating tissue
- Lessens patient pain during dressing changes and removal
- Maintains moist wound environment for healing
- Indicated for both partial and full thickness wounds
- Easily cut to size

The many applications for Tegaderm™ Contact Layer include:

Chronic Wounds



• Vascular ulcers



• Pressure ulcers



• Cancerous lesions

Donor Sites, Second Degree Burns and Graft Fixation




• Donor sites



• Graft Fixation

Surgical Wounds

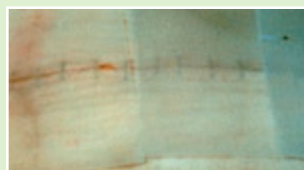

• Incisions

Traumatic Wounds



• Abrasions




• Lacerations

3M™ Tegaderm™

3M™ Tegaderm™ Non-Adherent Contact Layer

Application and Removal Guide

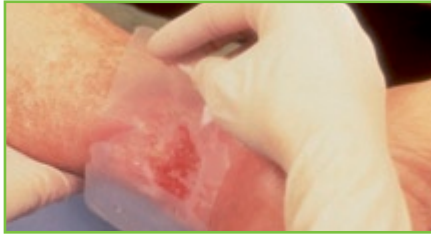

Position Tegaderm™ Contact Layer over wound.



If absorbent filler is needed, fill with 3M™ Tegaderm™ High Integrity Alginate Dressing or other suitable dressing. If moisture is needed for drier wounds, use 3M™ Tegaderm™ Hydrogel Wound Filler.




Dress wound with appropriate secondary dressing such as 3M™ Tegaderm™ Foam Dressing (non-adhesive), a 3M™ Tegaderm™ Transparent Film Dressing, gauze or other suitable dressing. Secure secondary dressing as appropriate.



Tegaderm™ Contact Layer may be applied under 3M™ Coban™ 2 Layer Compression System or 3M™ Coban™ Self-Adherent Wrap.



Remove Tegaderm™ Contact Layer by gently lifting. If wound surface appears dry, first moisten material with sterile saline.

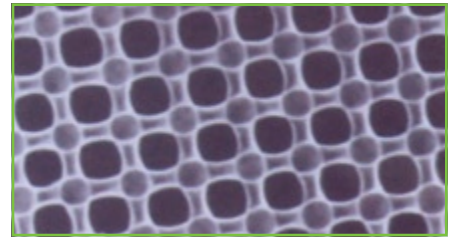

3M™ Tegaderm™ Contact Layer magnified 40x.

Ordering Information:

3M™ Tegaderm™ Non-Adherent Contact Layer

Catalog No.	NDC No.	Size	Dressings/Box	Boxes/Case	HCPCS Code*
5642	8333-5642-01	3 in. x 4 in. 7,5 cm x 10 cm	10	4	A6206
5643	8333-5643-01	3 in. x 8 in. 7,5 cm x 20 cm	10	4	A6207
5644	8333-5644-01	8 in. x 10 in. 20 cm x 25 cm	5	4	A6208

*HCPCS Codes are referenced in DMERC Surgical Dressing medical Policy.

Disclaimer:

HCPCS Codes have been provided to assist you in the preparation of Medicare Part B (DMERC) claims. Please note, however, that the reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.

3M™ Tegaderm™ Non-Adherent Contact Layer, formerly known as 3M™ Tegapore™ Wound Contact Material, is a new member of the expanding family of Tegaderm™ Brand Dressings.

3M™ Tegaderm™ Dressings **SIMPLE. DEPENDABLE. TRUSTED.**

For more information, contact your 3M Skin Health representative or call the 3M Health Care Customer Helpline at **1-800-228-3957**. Outside of the United States, contact the local 3M subsidiary.



Health Care

3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
USA
1-800-228-3957
www.3M.com/healthcare

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-800-563-2921

3M, Coban, Tegapore and Tegaderm are trademarks of 3M.
Printed in U.S.A.
© 3M 1994. Portions Copyright 2006. All Rights Reserved.
70-2008-4840-9 (116.5) DPI

Please Recycle